

Version: 1.0

## ALCODEX

### Fast disinfectant for surfaces



**3 ml, 100 ml 1 L, 5 L**

**Active substances:** Ethanol 96% - 75%  
Quaternary ammonium compounds – 0,5%

#### **Properties:**

ALCODEX is an aldehyde-free disinfectant for surfaces of medical devices. Removes blood, proteins, secretions and grease from all surfaces and medicinal instruments and can be rinsed off quickly, leaving no residue. The product has also an effect on those bacteria that become rapidly resistant to ordinary disinfecting agents. Suitable for using on all alcohol-enduring surfaces. ALODEX has a wide fungicidal and bactericidal effect, destroys Gram-positive and Gram-negative bacteria (including the generator of tuberculosis), deactivates viruses (HBV, HIV, Rota). pH 6.5-7.5

#### **Directions for use:**

ALCODEX is used as a concentrate. Do not dilute! If endurance of a surface is not known, it can be tested on an inconspicuous spot of the surface. It is not suitable for using on acryl glass and alcohol-solved varnish. The preparation is sprayed on the surface and wiped over it. The surfaces have to be covered in full. The time of effect is 1 – 3 minutes. Compatibility with aldehyde-containing preparations is not given. For this reason, aldehyde containing products should not be used before or after reprocessing with ALCODEX.

**Warnings and precautions:**H225: Highly Flammable liquid and vapour.

H319: Causes serious eye irritation.

H410: Very toxic to aquatic life.

P102: Keep out of reach of children.

P210: Keep away from heat/sparks/open flames/hot surfaces. No smoking.

P233: Keep container tightly closed.

P264: Wash skin thoroughly after handling.

Version: 1.0

P280: Wear protective gloves/protective clothing/eye protection/face protection.

P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313: If eye irritation persists: Get medical advice/attention.

P273: Avoid release to the environment.

P501: Dispose of contents/container in accordance with local regulations.

**Storage:**

Store below 25 °C in original package, protected from direct sunlight.

**Reg. no:**

**Manuf. date:**

**Expiry date:**

**Manufactured by:**

Interchemie Werken De Adelaar Eesti AS, Vanapere tee 14, Püünsi, Viimsi vald, 74013, Harjumaa, Estonia  
Tel.: +372 6 005 005